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Askina® Heel Sterile non adhesive hydrocellular heel dressing

1. PRODUCT DESCRIPTION

Askina® Heel is a sterile, non adhesive hydrocellular heel dressing that includes a polyurethane foam wound contact surface with high absorption capacity and a vapour permeable, water and bacteria resistant polyurethane film outer layer. In the presence of exudate, Askina® Heel helps maintain a moist wound environment conducive to natural healing conditions. Askina® Heel protects the heel area from pressure and pressure ulcers, and reduces pressure from external forces.

2. INDICATIONS

Askina® Heel is indicated for the management of moderately to heavily exuding, partial to full thickness wounds on the heel, such as:

- Stage I – IV pressure ulcers
- Surgical incisions
- First and second degree burns.

Askina® Heel may also be used as an aid for the prevention of skin breakdown on the heel.

3. CONTRAINDICATIONS

Askina® Heel is contraindicated for:

- Ulcers resulting from infections, such as tuberculosis, syphilis, deep fungal infections;
- Bites
- Third degree burns.
- In case of infection with inflammatory signs (temperature, oedema, redness, pain) contact proper medical authority. Resume use of Askina® Heel when normal healing conditions are present again. Do not use if pack is damaged or opened.

4. INFORMATION FOR USE

Askina® Heel is very simple to apply, requiring no special skills or equipment. The interval between dressing changes will depend entirely upon the state of the wound. On heavily exuding wounds, daily changes may be required at the beginning of treatment but this may be reduced to every 2 to 3 days for low exuding or epithelializing wounds.

4.1 Skin Preparation

- a) Clean the wound using a Prontosan® solution or in accordance with the accepted protocol.
- b) Dry the skin surrounding the wound.

4.2 Dressing Application

Apply Askina® Heel to the wound. Askina® Heel may be held in place using the securing strips (foam strap and two hook straps) provided or may be fixed in place with an appropriate secondary dressing e.g. fixation or pressure bandage. Adhesive tape may also be used, but care must be taken not to injure fragile skin on its removal.

4.3 Dressing Changes

Askina® Heel should be changed after 24 hours to 7 days, depending on the amount of exudate from the wound. Where leakage occurs the dressing should be changed immediately.

- a) Gently remove Askina® Heel
- b) Follow procedure 4.1 and 4.2 to apply a new dressing.

5. SPECIAL NOTES

The wound may initially appear to increase in size in the early stages of Askina® Heel treatment. This is normal and occurs as any wound debris is removed from the edges of the wound. This clears the way for healing.

In the management of moderately to heavily exuding wounds, Askina® Heel can make the overlying environment more conducive to healing. There are cases where healing is impaired as a result of underlying conditions; in these instances, Askina® Heel alone may make little or no progress, and suitable treatment of the underlying conditions will be necessary as well. Therefore, if after 4-6 weeks of Askina® Heel treatment, there has been no improvement then, in line with accepted wound management practice, the original diagnosis and overall therapy should be reassessed.

Askina® Heel should be left in place as long as possible in order to prevent trauma to the fragile newly formed tissue and to reduce cross contamination through frequent dressing changes.

Thick necroses should be removed before applying Askina® Heel.

6. STORAGE / STERILISATION / SYMBOLS ON LABELLING

Store dressings away from direct sunlight (☀) at ambient temperature and humidity (☂)

STERILE R = Gamma Sterilized

 = Do not re-sterilize

 = For single use only

LOT = Batch number

REF = Product code

 = Attention, see instruction for use

 = Use by: year and month

 = Manufacturer

 = Russian Safety Standards (Gost-R)

7. PRESENTATION

Askina® Heel is available in a box of 5 units.

WARNING:

Re-use of single use devices creates a potential risk to the patient or user. It may lead to contamination and/or impairment of functional capability. Contamination and/or limited functionality of the device may lead to injury, illness or death if the patient. Sterile unless pack damaged or opened.

Manufacturer: B.Braun Hospicare Ltd. Collooney, Co. Sligo, Ireland
Importer and distributor: CWT – Chronic Wound Treatment Ltd.
84 Hagefen St. Moshav Gamzo 73130 / 08-9287778 / www.cwt.org.il
Authorized by the Ministry of Health Medical Device # 24360004

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