Askina® DresSil, Askina® DresSil Border, Askina® DresSil Heel & Askina® DresSil Sacrum

INSTRUCTIONS FOR USE

PRODUCT DESCRIPTION

Askina® DresSil, Askina® DresSil Border, Askina® DresSil Heel and Askina® DresSil Sacrum are sterile dressings consisting of

- a perforated silicone wound contact layer
 - a highly absorbent polyurethane foam
 - a vapour permeable waterproof outer film
- In the presence of exudate, Askina® DresSil, Askina® DresSil Border, Askina® DresSil Heel and Askina® DresSil Sacrum help maintain a moist wound environment conducive to natural healing conditions.

INDICATIONS

Askina® DresSil, Askina® DresSil Border, Askina® DresSil Heel and Askina® DresSil Sacrum are indicated for the management of moderately exuding, partial to full thickness wounds, such as: pressure ulcers; venous and arterial leg ulcers; diabetic foot ulcers; first and second degree burns; donor sites; skin tears; and difficult to dress areas.

CONTRAINDICATIONS

Do not use Askina® DresSil, Askina® DresSil Border, Askina® DresSil Heel or Askina® DresSil Sacrum

- on ulcers resulting from infection, such as tuberculosis, syphilis, or
- deep fungal infections, or on
- third degree burns
- If signs of infection develop or if the wound deteriorates unexpectedly, contact the proper medical authority.

Resume use when normal healing conditions are present

INSTRUCTION FOR USE

- 1) Cleanse the wound with an appropriate solution (e.g. Prontosan) in accordance with normal procedures.
- 2) Dry the surrounding skin.
- 3) Select the appropriate size to cover the wound surface, ensure a 2 to 3 cm margin of foam beyond the wound edge.
- 4) Remove one of the backing liners and apply the adherent side to the wound.
- 5) Smooth the dressing over the wound removing the remaining backing liner.
- 6) To position Askina DresSil Heel, follow fig. 1

For venous leg ulcers, compression therapy may be used in conjunction with Askina® DresSil and Askina® DresSil Border when so directed by a physician. For difficult to dress areas additionally secure with tapes or bandages.

DRESSING REMOVAL/CHANGES

The interval between dressing changes will depend entirely upon the state of the wound and the amount of exudate.

Askina® DresSil, Askina® DresSil Border, Askina® DresSil Heel and Askina® DresSil Sacrum should be changed when it is saturated with exudate : 2 to 3 days on average or every 24hrs when the amount of exudate is significant. Change immediately if leakage occurs. Use rinsing solution to facilitate removal when necessary.

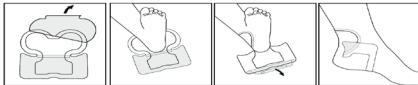
Figure 1

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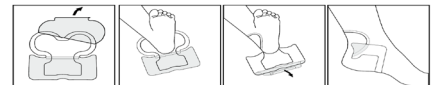
SPECIAL NOTES

The wound may initially appear to increase in size in the early stages of treatment. This is normal and occurs as any wound debris is removed from the edges of the wound. This clears the way for healing.

There are cases where healing is impaired as a result of underlying conditions; in these instances, Askina® DresSil, Askina® DresSil Border, Askina® DresSil Heel or Askina® DresSil Sacrum alone may make little or no progress, and suitable treatment of the underlying conditions will be necessary as well. If after 4-6 weeks of Askina® DresSil, Askina® DresSil Border, Askina® DresSil Heel or Askina® DresSil Sacrum treatment, there has been no improvement, then, in line with accepted wound management practice, the original diagnosis and overall therapy should be reassessed.

 = Russian Safety Standards (Gost-R)

PRESENTATIONS

Askina® DresSil is available in the following sizes

- 5 x 7 cm
- 10 x 10cm
- 15 x 15cm
- 10 x 20cm
- 20 x 20cm

Askina® DresSil Border is available in the following sizes:

- 6 x 6 cm
- 7,5 x 7,5cm
- 10 x 10cm
- 15 x 15cm
- 15 x 20cm
- 20 x 20cm

Askina® DresSil Heel is available in the following size

- 22 x 21,6cm

Askina® DresSil Sacrum is available in the following size:

- 16 x 17,5cm (225cm²)
- 18 x 20cm

Warning: Re-use of single-use devices creates a potential risk to the patient or user. It may lead to contamination and/or impairment of functional capability. Contamination and/or limited functionality of the device may lead to injury, illness or death of the patient. Sterile unless pack damaged or opened

 Store in a cool dry place

 Away from direct sunlight

 5°C > 25°C

 Ethylene oxide sterilised

 Do not Resterilize

 Single use only

 Batch number

 Product code

 Attention, see instructions for use

 Use by: year and month

 Manufacturer

 Do not use if pack is damaged

 Recyclable

Manufactured by: B. Braun Hospicare Ltd. Collooney, Sligo, Ireland
Importer: CWT – Chronic Wound Treatment Ltd.,
84 Hagefen st., Moshav Gimzo.

08-9287778, info@cwt.org.il, www.cwt.org.il

Authorized by the Ministry of Health. Medical

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