Limbo Waterproof Protectors

Indication: Protection from penetration of liquids into plaster casts and dressings.

Contraindication: At the time of production of this leaflet, none were known

Instructions for use.

Please read before using your Limbo and retain for future reference.

Fitting your Limbo

Pull the Limbo over your cast, gripping it by the edge of the seal as shown, and pull it up as high as you can.

Once the Limbo is as high as you can get it, gently tug it downwards and the seal will then lie flat against your skin.

On leg models fit a thick sock or similar over the cast foot before fitting the Limbo. This provides cushioning and reduces the risk of damaging the Limbo.

If the Limbo is to be submerged, especially leg models in the bath, we recommend that any air trapped in the Limbo is removed. This is done when it is out of the water by carefully lifting the seal and squeezing the air out of the Limbo and then letting the seal go back against the skin.

The Limbo should be worn for no longer than approx 20 minutes as the trapped air gets quite warm which can cause perspiration and discomfort.

Removing your Limbo.

Carefully roll the seal back on itself as shown in the illustration so that its inner surface faces outwards. This action prevents removal of the dressing. Afterwards slide the Limbo off, by pulling it gently and carefully remove it.

Storage and Care of your Limbo

- Handwash as required with warm soapy water.
- Avoid folding the seal.
- Hang upside down and air dry before storage.
- Ensure seal is undamaged before re-use

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Product Safety Instructions.

General medical advice is that swimming or similar vigorous water activities (e.g. water slides etc.) should be avoided during the healing process and if you are in any doubt you should consult a medical practitioner.

The product can be used for gentle submersion, e.g. in a bath or 'splashing about' in a pool provided the following safety procedures are followed:

Activities such as wading/paddling etc can be done, **ALWAYS WITHIN ONE'S DEPTH - CHILDREN MUST BE SUPERVISED AT ALL TIMES.**

Never enter a pool unsupervised.

Care must be taken not to tread on any sharp objects such as rocks or stones.

The LimbO can be used in hydrotherapy as part of your medically approved treatment.

Special note: Full leg casts are heavy and severely restrict movement, so for safety reasons you should not go into pools or the sea.

Use the LimbO according to these instructions for safe use. The manufacturer will not be responsible for any matters resulting from misuse or incorrect use of the product. The LimbO is guaranteed to last for the entire period of the treatment - up to two months of continuous use, when properly used.

The product is available in a wide range of models and sizes for both children and adults

100% latex-free.

Manufacturer: Thesis Technology Products Ltd.

Manufacturer's Address: Green Farm Brooks, Brooks Lane, Bosham, West Sussex, United Kingdom.

Importer and Registration Holder: CWT - Chronic Wound Treatment Ltd.

Importer's Address: 84 HaGefen St., Moshav Gamzu. 08-9287778. www.cwt.org.il

Authorized by the Ministry of Health

Ministry of Health Medical Devices number 24360026

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